

**eyeeco**  
Product Catalog



# tranquileyes™

Eye Hydrating Therapy



## **Hydrated eyes are beautiful, radiant eyes.**

Tranquileyes eye hydrating therapy is a revolutionary, doctor recommended system that does for the eyes what the best skin care products do for skin, hydrate. Proven effective, tranquileyes helps your eyes feel and look their best. Use the system to hydrate dry, tired eyes, reduce eye puffiness and create complete darkness to enhance relaxation and sleep.





## What is tranquileyes™?

**Use the system to hydrate dry, tired eyes, reduce eye puffiness and create complete darkness to enhance relaxation and sleep.**

### **Tranquileyes for Relief**

Over 50 million people suffer from symptoms of dry eye, ranging from mild to severe. The most common cause of eye dryness is normal aging. However, many lifestyle and environmental conditions including contact lens wear, computer use, air travel, allergies, LASIK and medical conditions such as Sjogren's syndrome, further challenge our ability to keep eyes well hydrated. Tranquileyes eye hydrating therapy brings immediate and long lasting relief to dry, irritated eyes. While there is no cure for dry eyes, tranquileyes help the 10 million people suffering from chronic dry eyes by enhancing existing dry eye treatments such as artificial tears, ointments, supplements, Restasis™ or punctal plugs. For the other 40 million suffering from mild dry eye symptoms, tranquileyes can comfortably help prevent chronic dry eyes from developing.

### **How tranquileyes Work**

Use tranquileyes Moist Heat Therapies to relieve dry eyes. As we age, our bodies produce less oil, 60% less at age 65 than at age 18. This oil deficiency, which is more pronounced in women, affects the skin as well as the tear film. Additionally, many environmental, lifestyle and physical conditions clog the glands that release these critical oils. With less oil to seal the watery layer, the tear film evaporates much faster, leaving eyes dry and irritated. Tranquileyes soft, flexible goggle forms a warm, moist humid environment around the eyes that help stimulate tear production and prevent the evaporation of tears. The heat stimulates and loosens the critical oils that slow evaporation of natural tears. The warm moisture increases the relative humidity around the eyes to prevent the evaporation of tears while moisturizing the delicate surrounding skin. The results are clear, comfortable eyes that you can see and feel.

Create Cold Therapies with tranquileyes to help reduce eye puffiness resulting from eye dryness (the sensitive skin covering the eyes rubbing over a dry surface), lack of sleep, allergies or having too much fun.

Enjoy the added benefit of complete darkness while using tranquileyes to promote deep sleep or relaxation.

## The Original tranquileyes™ Eye Hydrating Therapy

**Use warm or cold water to create mild Moist Heat or Cold Therapies. All goggles create complete darkness to enhance sleep or relaxation.**

Treatments are created by soaking the moisture pad (tucked in the back of the goggle) in warm or cold water. Tranquileyes are latex free and made in the United States.



### Mini

Includes one tranquileyes goggle with manual, packaged in a travel size canister.



### Basic

Includes one tranquileyes goggle, one spare set of foam replacements, net dry bag and manual.



### Deluxe

Includes one tranquileyes goggle, four spare sets of foam replacements, net dry bag, miniature stainless steel can for travel, 1.0 fl. oz. Cleanser for Foam, and manual.



### Chronic Dry Eye Kit

Includes one tranquileyes goggle with foam for sensitive skin, two spare sets of foam replacements, net dry bag, 1.0 Fl. Oz. Cleanser for Foam, travel sack.

## thermoeyes

### Enhance Moist Heat and Cold Therapies with thermoeyes gel packs and our botanical waters.

#### Thermoeyes Reusable Gel Packs

Thermoeyes are optional gel packs that boost the original tranquileyes therapies by increasing temperature intensity. Thermoeyes tuck inside the goggle, replacing the thick gray moisture pad used for mild warm/cold treatments. A thinner white foam pad (thermopad) is soaked in warm (dry eye treatments) or cold water (puffy eye treatments) and placed on top of the gel pack.



#### Blue Instant Version

The blue instant version can be used to boost the original warm and cold therapies. For extra heat, snap the stainless steel disc inside the packet and the thermoeyes instantly generate heat. The heat lasts approximately 30 minutes. To reset after use, boil continuously in water for several minutes, remove and let cool. To boost Cold Therapy, place gel packs in freezer or refrigerator for ten minutes. Thermoeyes can be reset and used approximately 100 times for heat and indefinitely for cold. Cannot be placed in microwave.



#### Green Microwaveable Version

For extra heat, place gel packs in a cup of water and place in microwave. Once warm, remove and insert in goggle with warm thermopad. Not recommended for boosting cold treatment.

#### How to Use Blue thermoeyes (Instant Version)



Snap internal disc



Insert into eye cover with moist pad



Simply boil when done to reset

## Botanical Waters

### Relax, Puffy Eye and Age Defense

Enhance the benefits of tranquileyes moist heat and cold therapies with one of three specially formulated botanical water formulas. Simply mist over the face or over moisture pads prior to wearing the tranquileyes goggle. Each water formula addresses a specific benefit. All include the active ingredient of Allantoin and Aloe to aid in wrinkle relief, while moisturizing the entire eye area.



#### Relax Formula (Lavender)

Relax Formula is infused with soothing Lavender essential oil to calm your mind and ease stress.



#### Puffy Eye Formula (Cucumber)

Puffy Eye Formula is infused with the moisturizing and anti-inflammatory benefits of Cucumber and Aloe.



#### Age Defense Formula (Honeysuckle and Green Tea)

Age Defense Formula combines Green Tea, Vitamin C, and Honey-suckle to help protect skin,

## All-Inclusive Kits

**Our best-selling, all-inclusive TE kits include everything a user needs to receive the full benefits of the tranquileyes system.**



### **Water Therapy TE Kits**

The Water Therapy TE kits include one tranquileyes goggle, one set of Blue Instant thermoeyes, net dry bag, travel sack and one 2.0 fl. oz. botanical water formula. The Relax TE Kit features our Relax Formula (lavender), the Puffy Eye TE kit features the Puffy Eye Formula (cucumber), and the Age Defense TE kit features the Age Defense Formula (honeysuckle, green tea and vitamin C).



### **Chronic Dry Eye TE Kit**

The Chronic Dry Eye TE kit features our foam for sensitive skin, especially designed for chronic dry eye sufferers.

The kit includes one tranquileyes goggle, one spare set of foam replacements, one set of Blue Instant thermoeyes, 1.0 fl. oz. Foam for Cleanser, net bag for drying and travel sack.

## Gift Sets and Accessories



### **Tranquileyes Gift Set with Thermoeyes**

These beautiful, quality purses are an ideal gift. Each bag has a black handle with zipper top. The gift set includes one tranquileyes goggle in color of choice, two botanical waters of choice, one set of Blue Instant thermoeyes, stainless steel travel tin, net bag for drying and manual.

## Tini Bikinis



### Tini Bikini Gift Set

Fun and functional, Tini Bikinis wrap around the tranquileyes goggle and tuck inside the back, protecting the foam from lotions or makeup. Three snaps, secure the bikini in place. All Bikinis are hand sewn in California and are machine washable. The Tini Bikini Gift Set includes one tranquileyes goggle with one tini bikini and a manual.

### Tini Bikini Covers

The Tini Bikinis are also sold separately. They are available in a variety of colors and patterns.



Pink



Coffee



Tigris



Gold Lame'



Blue Polka Dot



Catalina



Maui



Cancun



Tahiti



Fiji



Buenos Aires



Santa Barbara

## Features and Benefits



### **tranquileyes**

Tranquileyes have beautifully sculpted, anatomically designed eye covers that are made from medical grade, flexible rubber material that is FDA approved. The goggles seal only the area required to control the desired environment, delivering complete darkness and exceptional comfort. The eye covers rest on the orbital bones, avoiding any restriction of blood flow to vital arteries above the nose and do not place unnecessary pressure on the sensitive blood vessels under the eye. All eye covers are Latex free.

### **Eye Cushions**

Adhered to each eye cover are comfortable eye cushions made from visco-elastic foam. The foam is specially designed to form to the face when warmed by body heat and provide increased comfort while creating a complete seal of the orbital area. This seal helps to control the temperature and humidity around the eyes and surrounding area. All foam is Latex free.

### **Moisture Pads**

Placed in each eye cover is a small pillow of visco-elastic foam called a moisture pad. The moisture pads can be removed and soaked in warm or cold water for desired result. Warm water creates the Moist Heat Therapy for relieving dry eyes and cold water creates the Cold Therapy for helping to reduce puffiness. All foam is Latex free.

### **Comfort Wrap**

A uniquely designed comfort wrap holds the eye covers in place. The comfort wrap is wide in the back and narrows toward the temples for maximum comfort. The wrap is easily adjusted with two specially designed sliders, to ensure a custom fit and complete seal of the tranquileyes to the eye area.

## Maintenance



### Sensitive Skin Foam Replacements

Our sensitive skin foam is made of visco-elastic memory foam and does not have a fabric lining. Foam replacements are available in sets of 3 and sets of 6. Each set includes both the left and right eye cushions and moisture pads (4 parts per set). Each set lasts up to 60 uses.



### Standard Foam Replacements

Our standard foam is made from visco-elastic memory foam and is lined with a soft, black Lycra fabric on the top. Foam replacements are available in sets of 3 and sets of 6. Each set includes the both the left and right eye cushions and moisture pads (4 parts per set). Each set lasts up to 60 uses.



### Spare Head Wraps

Colorful spare head wraps are available in black, brown, pink and turquoise.

### Cleanser For Foam

Our Cleanser For Foam is a gentle anti-bacterial soap that can be used to wash the foam as well as the entire goggle as needed. Available in a 1.0 fl. oz. size.



### Net Dry Bag

Allows for easy cleaning and drying. Available in white.

### Travel Sack

Our travel sacks are made with anti-static polyester fabric and have a drawstring closure. Available in gray.



## Methods of Using tranquileyes



### How to Use tranquileyes:

Dampen moisture pads in warm (Moist Heat Therapy) or cold water (Cold Therapy), squeeze out excess moisture and return to eye covers. Optionally mist one of the Water Formulas over moistened pads and/ or face. Wear at least 20 minutes. For severe dry eyes, apply artificial tears or ointments to eyes as directed prior to putting tranquileyes goggles on. Wear for at least 45 minutes. Tranquileyes are safe to wear all night. Further enhance all therapies using thermoeyes reusable gel packs.

### Moist Heat Therapy Benefits:

Relieves eye dryness and fatigue caused by many physical, environmental and lifestyle conditions. Naturally helps stimulate tear production and restores vital moisture by helping to release critical oils from the meibomian glands and increasing humidity around the eye. The heat releases the oils that slow natural tear evaporation and the increased humidity prevents the tears from evaporating. The increased humidity also hydrates surrounding eye area to help minimize and smooth wrinkles.

### Cold Water Therapy Benefits:

Helps reduce eye puffiness caused by dryness, allergies, lack of sleep or a good cry. Also helps relieve migraine and sinus headaches. The Cold Water Therapy soothes and refreshes eye area by helping to reduce swelling around the eyes.

The benefits of tranquileyes are cumulative. The more you use tranquileyes, the better the results. Tranquileyes provide the added benefit of creating complete darkness which helps stimulate the pineal gland, releasing melatonin to relax the body and improve your ability to sleep soundly.

### Recommended Use

- Hydrate dry eyes  
Moist Heat Therapy + tranquileyes Waters (optional) for 20 minutes daily
- Hydrate chronic dry eyes  
Moist Heat Therapy + thermoeyes + supplements + artificial tears and/or ointments for 45 minutes to 8 hours daily
- Reduce puffiness around eyes  
Cold Water Therapy + Puffy Eye Formula, for 20 minutes, as needed
- Minimize wrinkles around eyes  
Moist Heat Therapy + Age Defense Formula for 20- 30 minutes, daily
- Relieve sinus and migraine headaches  
Cold Water Therapy for 45 minutes or as needed
- Offset symptoms of jet lag  
During flight, moisten foam pads with Relax Formula. Wear 10 minutes per hour for maximum comfort. Before or after flying, use Moist Heat Therapy + Relax Formula and wear 1 – 8 hours.
- Sleep aid  
Warm Water Treatment + Relax Formula, wear for 1 – 8 hours.
- Reduce stress  
Warm Water Treatment + Relax Formula, wear for 30 min. to 1 hour as needed

\*\* USE THERMOEYES TO ENHANCE TO BOOST AND ENHANCE ANY OF THE ABOVE THERAPIES.



Eye Eco, Inc. [www.eyeeeco.com](http://www.eyeeeco.com)  
APAC International, Inc (International Distributor)  
[www.apac21.com](http://www.apac21.com) email: [info@apac21.com](mailto:info@apac21.com)  
Tel: (425)337-7021 Fax: (425) 367-4024  
Made in the U.S.A. Patented Technology with Patents Pending